


Drilling Sequence 485 Drill

TSIII | **SSIII** | **USIII**


(Length : 7mm)

Ø4.0mm

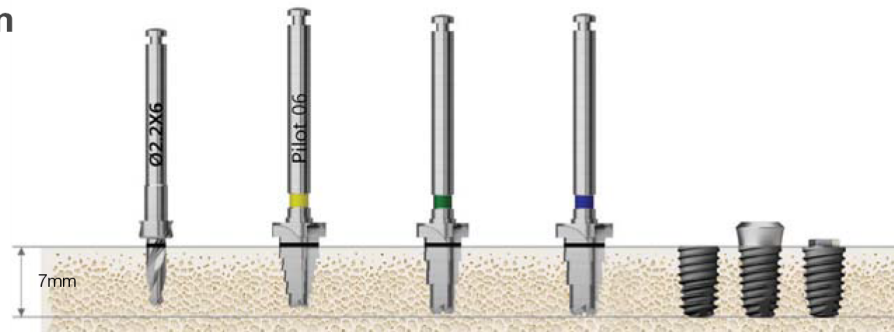


Bone Quality	Ø 2.2 Drill	Pilot Drill	F4.0 485 Drill	F4.5 485 Drill	Ø 4.0 Fixture
Normal	▶	▶	▶		Implant Placement
Hard	▶	▶		▶	

Ø4.5mm

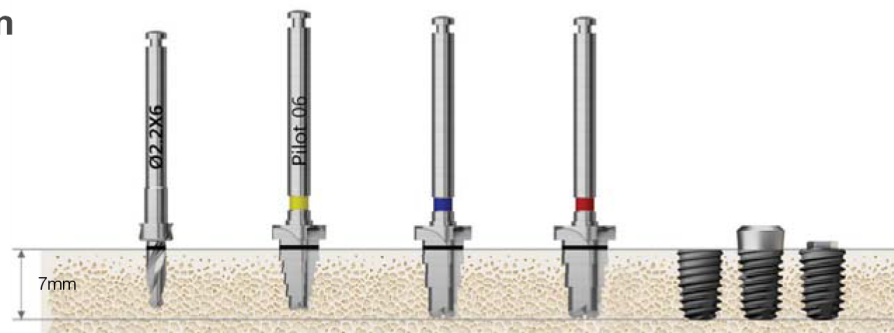

Bone Quality	Ø 2.2 Drill	Pilot Drill	F4.5 485 Drill	F5.0 485 Drill	Ø 4.5 Fixture
Normal	▶	▶	▶		Implant Placement
Hard	▶	▶		▶	

Ø5.0mm

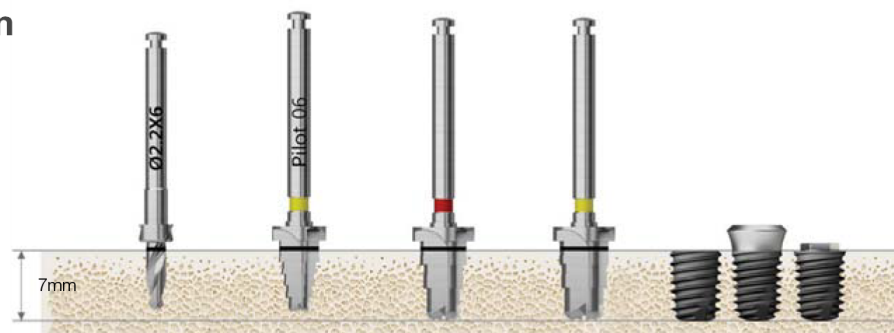

Bone Quality	Ø 2.2 Drill	Pilot Drill	F5.0 485 Drill	F5.5 485 Drill	Ø 5.0 Fixture
Normal	▶	▶	▶		Implant Placement
Hard	▶	▶		▶	