Bone Spreader KIT

User Manual

OSSTEM[®]
IMPLANT

OSSTEM[®]
IMPLANT

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4 sets of Spreaders
1 KIT cassette set

■ 4 sets of Bone Spreaders (OBOS22F, OBOS28F, OBOS35F, OBOS35R)

For the creation of space for incision, insert the spreader into the incised crestal bone and expand the bone in the incision in the forward or rotative direction.

■ KIT cassette 1 set

Made of corrosion-resistant stainless steel; boasts of thermal safety during sterilization

■ Product description

■ Length marking line

Marked in 7mm, 8.5mm, 10mm, and 11.5mm increments The Laser Marking between 10mm and 11.5mm is indicated as a thick line.

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■ Cleaning the KIT

Clean the treatment tools after use as follows :

1. After treatment, clean the tools using the conventional method, i.e. by soaking them in alcohol.
2. After cleaning the tools with distilled water or under running water, remove any remaining blood stain and foreign matter completely.
3. Remove moisture completely using a dry cloth or a heater fan.
4. Insert and secure the moisture-free treatment tools into the correct slots of the KIT.
5. After setting the tools, sterilize(132°C for 15 minutes) the KIT in an Autoclave and store at room temperature.

■ Instructions on Usage and Cleaning

1. Detach all used instruments after operation and clean, dry, and keep them under normal temperature.
2. Do not leave the instruments in a place vulnerable to contamination.
3. Be sure to sterilize these medical tools in Autoclave before use.(132°C for 15 minutes)
4. Do not use hydrogen peroxide for sterilization and cleaning(may damage the laser marking or cause discoloration).
5. This product comes with a ONE(1) year warranty from the date of its purchase.

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DEUTSCHE OSSTEM GmbH