


Drilling Sequence

MS Drill

Narrow Ridge | Denture | Provisional


(Length : 10mm)

Ø1.8mm

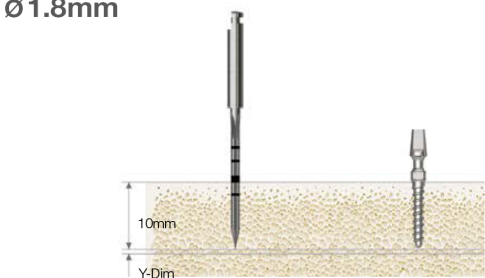


| Bone Quality | Lance Drill | Ø 3.0 Fixture     |
|--------------|-------------|-------------------|
| Soft         | ▶           | Implant Placement |
| Normal       | ▶           |                   |
| Hard         | ▶           |                   |

Ø2.0mm

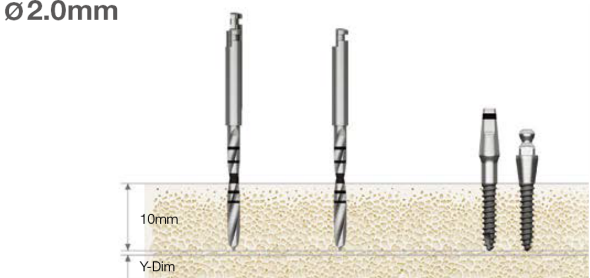


| Bone Quality | Lance Drill | Drill (Ø1.8) | Ø 2.0 Fixture     |
|--------------|-------------|--------------|-------------------|
| Soft         | ▶           | ▶            | Implant Placement |
| Normal       | ▶           |              |                   |
| Hard         | ▶           |              |                   |

Ø2.5mm

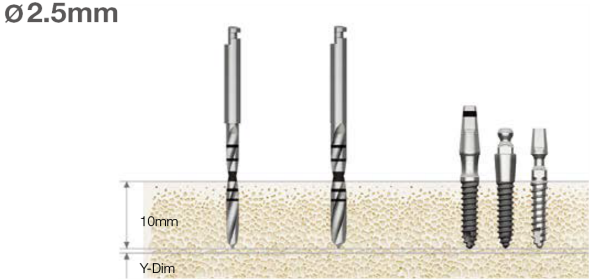

| Bone Quality | Drill (Ø1.8) | Drill (Ø2.3) | Ø 2.5 Fixture     |
|--------------|--------------|--------------|-------------------|
| Soft         | ▶            | ▶            | Implant Placement |
| Normal       | ▶            |              |                   |
| Hard         |              |              |                   |

Ø3.0mm

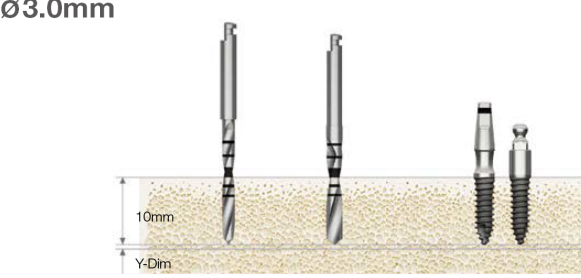

| Bone Quality | Drill (Ø1.8) | Drill (Ø2.5) | Ø 3.0 Fixture     |
|--------------|--------------|--------------|-------------------|
| Soft         | ▶            | ▶            | Implant Placement |
| Normal       | ▶            |              |                   |
| Hard         |              |              |                   |