


OneCAS Kit Drilling Sequence


Residual Bone : 7mm or more

Residual bone 4mm, soft bone: Ø4.0 Fixture x 8.5mm

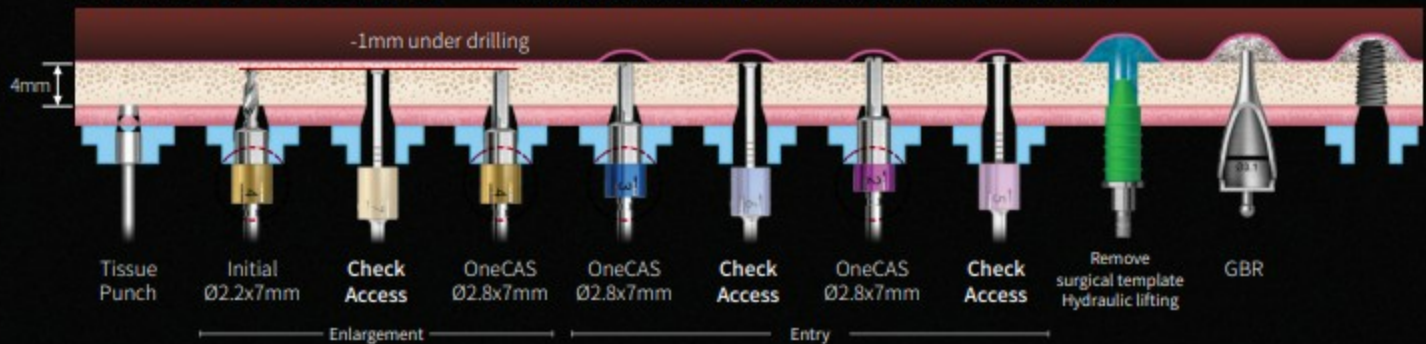


Residual Bone : 7mm - 9mm

Residual bone 8mm, soft bone: Ø4.5 Fixture x 10mm

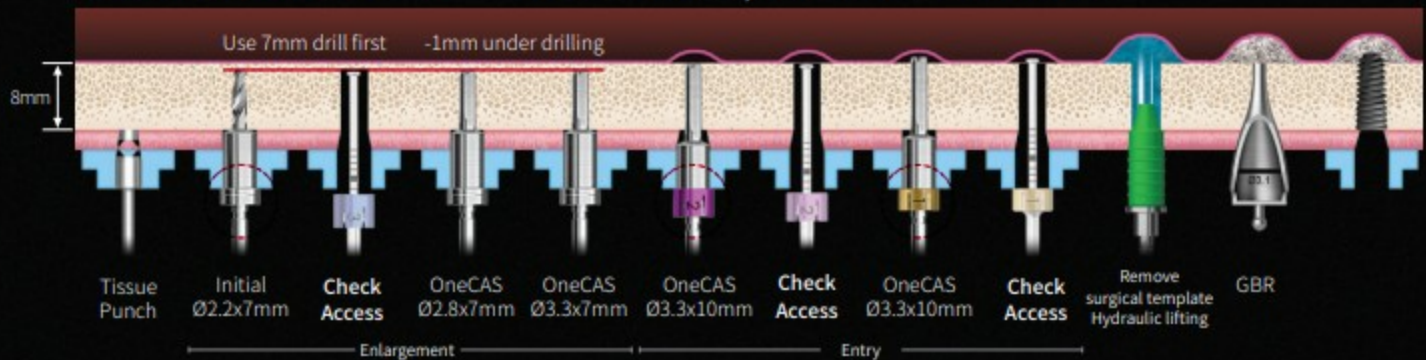

Residual Bone : 9mm - 10mm

Residual bone 9mm, soft bone: Place Ø5.0 Fixture x 10mm (*Guide hole: Ø5.8(W))

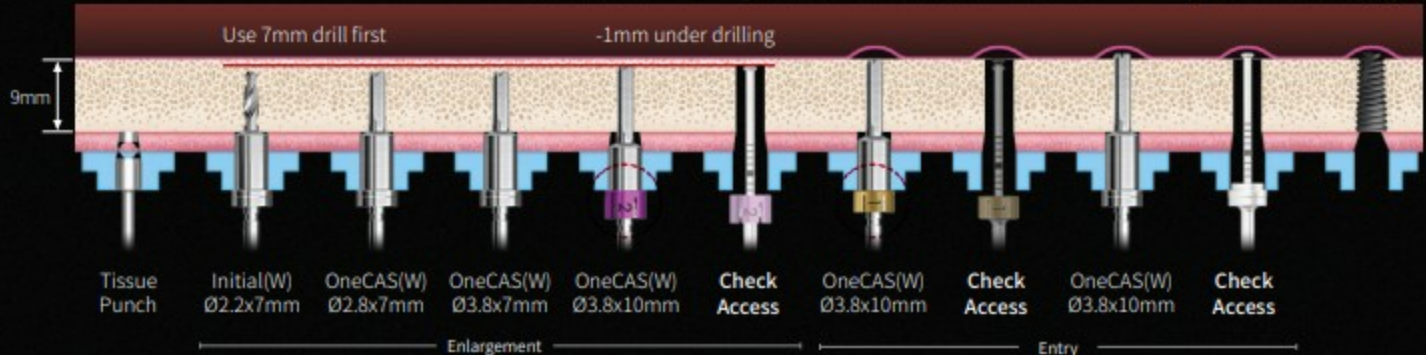

OneCAS drilling sequence (Only Enlargement)

Fixture	Bone	F2.2	F2.8	F3.1	F3.3	F3.6	F3.8	F4.1
Ø4.0	Soft	●	●					
	Normal	●		●				
Ø4.5	Soft	●	●		●			
	Normal	●		●		●		
Ø5.0	Soft	●	●				●	
	Normal	●		●				●