


Drilling Sequence **OneMS Drill**


TSIII(ø3.0) | MS
(Length : 10mm)

MS ø2.0

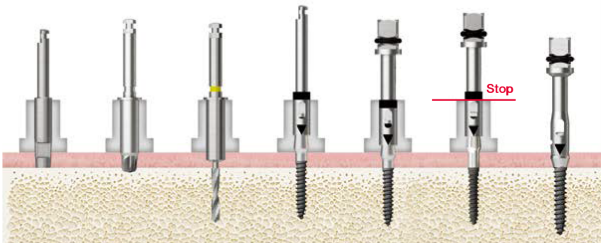


Bone Quality	Tissue Punch	Flattening Drill	OneMS Drill (ø1.5)	NoMount Driver	Fixture Driver		
					G/H 2.5	G/H 4.0	Denture
Soft	▶	(▶)	▶	▶		▶	
Normal	▶	(▶)	▶	▶		▶	
Hard	▶	(▶)	▶	▶		▶	

MS ø2.5

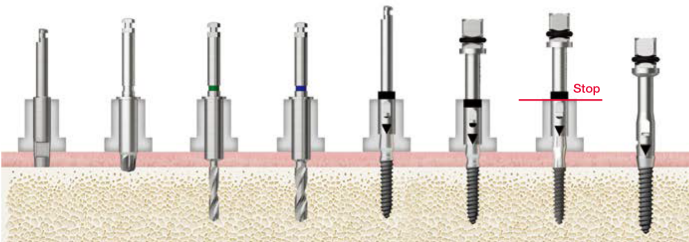


Bone Quality	Tissue Punch	Flattening Drill	OneMS Drill (ø1.8)	OneMS Drill (ø2.3)	NoMount Driver	Fixture Driver		
						G/H 2.5	G/H 4.0	Denture
Soft	▶	(▶)	▶	—	▶		▶	
Normal	▶	(▶)	▶	—	▶		▶	
Hard	▶	(▶)	—	▶	▶		▶	

MS ø3.0

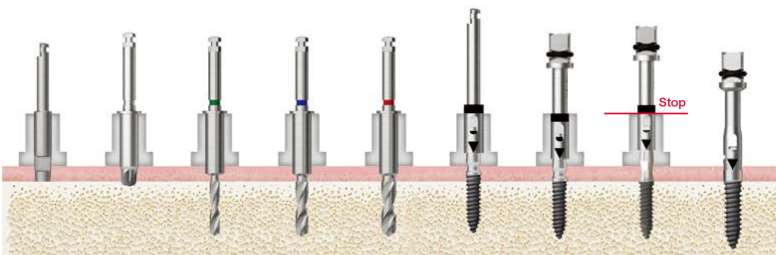

Bone Quality	Tissue Punch	Flattening Drill	OneMS Drill (ø1.8)	OneMS Drill (ø2.3)	OneMS Drill (ø2.7)	NoMount Driver	Fixture Driver		
							G/H 2.5	G/H 4.0	Denture
Soft	▶		▶	—	—	▶		▶	
Normal	▶		▶	—	—	▶		▶	
Hard	▶		—	▶	▶	▶		▶	

TSIII ø3.0

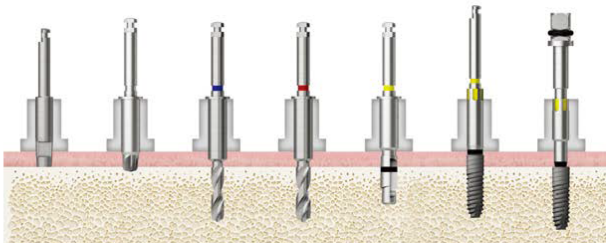

Bone Quality	Tissue Punch	Flattening Drill	OneMS Drill (ø2.3)	OneMS Drill (ø2.7)	F4.5 Cortical Drill	NoMount Driver	Fixture Driver
Soft	▶	(▶)	▶			▶	▶
Normal	▶	(▶)	▶	▶		▶	▶
Hard	▶	(▶)	▶	▶	▶	▶	▶

In case of Fixture 10 / 11.5 / 13mm sequence, precede number of drilling with 8.5mm Drill for each step
Ex. ø2.5×11.5mm MS Fixture
: Tissue Punch ▶ Flattening Drill ▶ **ø1.8×8.5mm** ▶ **ø1.8×11.5mm** ▶ NoMount Driver ▶ Fixture Driver